# WITNESS™ LH

# WITNESS™ LH

## CANINE AND FELINE SERUM LUTEINIZING HORMONE DETECTION KIT

WITNESS™ LH

English

### I. GENERAL INFORMATION

WITNESS™ LH test provides an accurate, semiquantitative measurement of canine and feline luteinizing hormone (LH) in serum samples. This assay, when used in conjunction with progesterone testing, identifies the pre-ovulatory LH surge, and thus, the time of ovulation. It may also be used to distinguish ovarioexcized from sexually intact bitches or queens.

### II. TEST PRINCIPLES

WITNESS™ LH is an immunochromatographic assay that uses gold-conjugated antibodies to give a visual line in the presence of luteinizing hormone in dog or cat serum. Sensitized particles bound to the luteinizing hormone present in the serum sample migrate along a membrane. The complex is then captured on a sensitized reaction zone, where its accumulation causes the formation of a clearly visible pink/red band. A control band located at the end of the reading window (3) ensures that the test was performed correctly. A positive result occurs when a line appears in the test area which is of similar or greater intensity than the control line. When this occurs, the LH level in the serum sample is greater than 1 nanogram per milliliter.

### III. SAMPLE COLLECTION

- The test can be performed on canine or feline serum. WITNESS™ LH requires only four drops of serum to run each test.
- Always collect samples with a sterile needle and syringe.
- The sample should not be haemolysed or lipemic.

### IV. SAMPLE STORAGE

Serum samples should preferably be tested on the same day as sample collection. If testing is further delayed, samples should be kept refrigerated (up to 24 hours at 2°C to 8°C). For long term storage, serum samples should be kept frozen (-20°C or colder). Do not thaw and refreeze.

### V. KIT CONTENTS

- 6 pouches, each containing a test device
- 6 pipettes
- Instructions for use

### VI. PRECAUTIONS

- Do not use this kit or any of its components after the expiration date.
- Kit should be stored at +2°C - 25°C. Kit should not be frozen.
- Use the test immediately after the pouch is first opened (within 10 minutes).
- Avoid touching or damaging the membrane in the sample well or the results window.
- The WITNESS™ device should be placed on a flat, horizontal surface while performing the test.
- Use a separate pipette for each sample.
- Hold pipette vertically when dispensing sample.
- Handle all samples as biohazardous material.
- For veterinary use only.

### VII. TEST PROCEDURE

**Prior to use, allow SAMPLE to come to room temperature (20° to 25°C).**

#### 1. SAMPLE APPLICATION

- Tear open a pouch provided and place the test device on a flat horizontal surface.
- Holding the provided pipette vertically, add four drops of sample to the test well (1).
- Leave the test device flat during migration of sample/reagent complex through the reading windows.

#### 2. READING TEST

- After 20 minutes, observe the presence or absence of pink/red bands in the reading windows (2) and (3).

### VIII. RESULTS

#### Valid Results

- Test is **VALID** if a pink/red band is present in reading window (3).

#### Interpretation of Results

**NEGATIVE.** If no line appears in reading window (2), the LH value is less than 1 nanogram per milliliter. Continue LH testing on a daily basis to determine the optimal time to breed.

**POSITIVE.** Field observations have shown if a line appears in reading window (2) which is of similar or greater intensity than the control window (3), the LH value is greater than 1 nanogram per milliliter. Field observations have shown if a distinct line appears in reading window (2) of less intensity than control window (3), the sample is likely to contain slightly less than 1 nanogram per milliliter and have been drawn before or after the LH surge (the shoulders of the LH peak). If the daily tests for the LH surge do not reveal a stronger test line, field observations have shown the weaker intensity line should be considered as indicative of the LH surge. Progesterone values may help confirm.

#### Note:

The test is complete and may be read before 20 minutes if both control and test pink/red bands are visible. The presence of only the control band on the strip before 20 minutes does not mean that the test is complete, as a test band may appear more slowly than the control band.

### SYMBOL DESCRIPTIONS

Use by Date

Authorized Representative in the European Community

Batch Code

Consult Instructions for Use

Serial Number

Manufacturer

Temperature limitation (storage temperature range)

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