

AlphaTrak3

BLOOD GLUCOSE MONITORING SYSTEM



Bluetooth® Setup Guide

1. Enable discovery mode on your meter



Press  for less than 3 sec to turn on your meter.



Press  again for less than 3 sec to view result history.




Press and hold both  and  together for 3 sec to toggle Bluetooth® discovery mode.


 will appear blinking on the screen.

While meter is off, press and hold "c" for more than 3 sec to enable/disable automatic Bluetooth discovery with every result.


2. Pair with your mobile

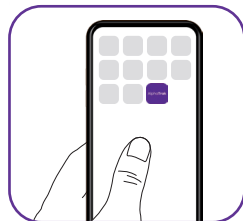


Open the **AlphaTrak** application.

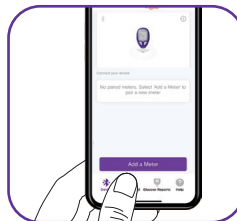

Tap **Device** at the bottom navigation. Tap **Add a Meter** to find your meter.

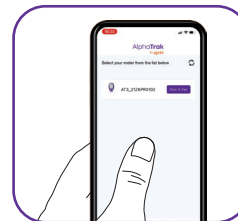

Tap  if you don't automatically find your meter. Select your meter and tap **Click to Pair**.

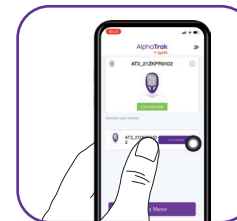

Enter the 6-digit pairing key, now displayed on your meter, to the application.

Application will confirm that you've successfully connected. On the meter, the Bluetooth® icon will stop blinking.

3. Import test data



Tap **Dashboard** at the bottom navigation.



Tap **Get Result** to get test results from your meter.



Application will confirm that test results successfully synced.

When getting test results, if the Bluetooth® mode is set to turn on automatically, you will see the blinking Bluetooth® icon once the result is ready. This indicates that Bluetooth® is on and ready to pair with the AlphaTrak App.



For additional information, please contact:

Product Support for Australia: 1800 814 883

<http://alphatrakinfo.com>

Zoetis Australia Pty Ltd. ABN 94 156 476 425. Level 6, 5 Rider Boulevard, Rhodes, NSW 2138. © 2022 Zoetis Inc. All rights reserved. February 2023. MM-24632

zoetis