





## Installing OSi Software onto i-STAT Alinity v Using a USB Drive

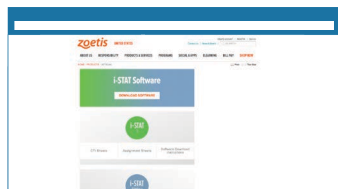

Go to:  
[www.zoetisUS.com/istatsoftware](http://www.zoetisUS.com/istatsoftware)  
> Select Download Software

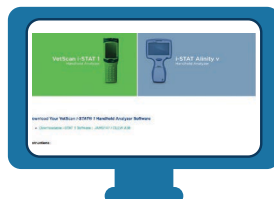


Navigate to Download Your i-STAT Alinity v Handheld Analyzer Software  
> Select Downloadable i-STAT Alinity v OSiXX System Software




Save software to a USB drive


*\* Requires FAT32 formatted USB 2.0 memory stick  
\* File must be saved on the top-level directory of the USB drive*

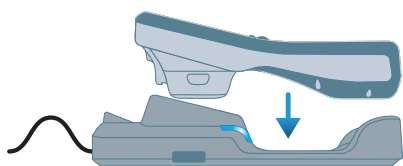

Place instrument into the Base Station and power on the instrument

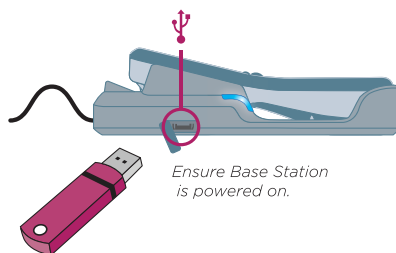

Insert USB drive into the USB port in the Base Station

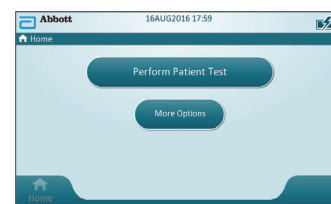


From the Home Screen:  
> More Options  
> Instrument Options  
> Software Installation  
> Install from USB




Follow prompts until transfer is complete

### SUCCESS

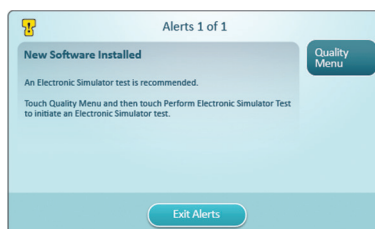

OSi Software Installation Results  
SUCCESS

### FAILURE

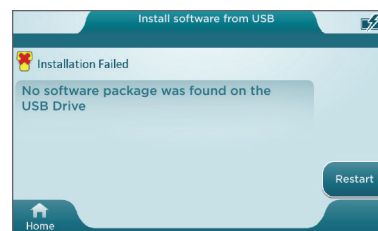

OSi Software Installation Results  
FAILURE

If additional assistance is required, please contact Zoetis Diagnostic Technical Support at 1-800-822-2947. Support is available 24 hours/day, 7 days/week.