



In-Depth Objective Cleaning

Should the objective become dirty as a result of faecal sample material or immersion oil, perform these steps to clean it:

1. **Power down** the scanner by pressing the power button
2. Remove the **power cable** and the **network cable** from the back of the **scanner**
3. Place a **protective cloth** on the **glass** beneath the **objective**
4. Clean the **objective** in place, using a **microfibre cloth** or **lens paper**. If necessary, try adding **warm distilled water** to the **cloth** or **using a cleaning swab/foam tip with lukewarm distilled water**
 - a. **IMPORTANT:** Never pour or spray any **liquids** directly on the **scanner**
5. If the **objective** does not come clean, carefully unscrew it.



Handle with Care: When handling the loose objective, do not touch either of the lenses to prevent dirt and damage.

6. Use the **microfibre cloth** and **warm distilled water** to clean the **lens**
 - a. **IMPORTANT:** Both the top and bottom objective lenses can be cleaned
 - Use ionised compressed air for cleaning the inner objective lens
 - When needed, use lens paper that is moistened using lukewarm distilled water.
 - If water is not sufficient for cleaning the residue on the lens, use a solution of 70% IsoPropyl alcohol and 30% distilled water to moisten the lens paper
 - b. **IMPORTANT:** Using acetone or xylene to clean the lens may damage it; however, if the lens is covered with glue/adhesive, cleaning the lens may require using stronger cleaners. Do not use cotton tipped applicator. Use swabs from the Starter Kit**

**If you are unsure about what cleaner to use, please contact Zoetis Diagnostic Support (1800 270 727)

7. Use a bulb syringe to remove any **dust** from the **lens** and **scanner**
8. Replace the **objective** by screwing it back in place
 - a. **IMPORTANT:** Do not forcefully screw the **objective** back onto the **scanner**
9. Remove the **cloth** covering the **glass**, replace the **power** and **network cables**, and turn the **scanner** on
10. Check the **scanner** by running a **test scan**

Zoetis Diagnostics Support:
0800 100 109 (dxsupport.nz@zoetis.com)