



# Quick User Guide For Citrated Whole Blood Testing

**InCoag**  
Automated Coagulation Analyser

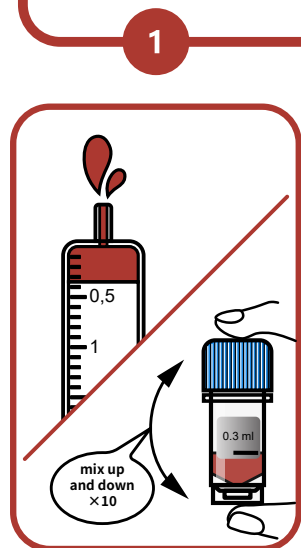

**Sample Preparation**

- ① Collect a minimum of 300  $\mu$ L of venous blood;
- ② Within 30 seconds, transfer the sample to the citrate tube;
- ③ Gently invert the tube 10 times to mix;
- ④ Citrated blood is stable for 1 hour at room temperature and up to 4 hours at 2-8°C.

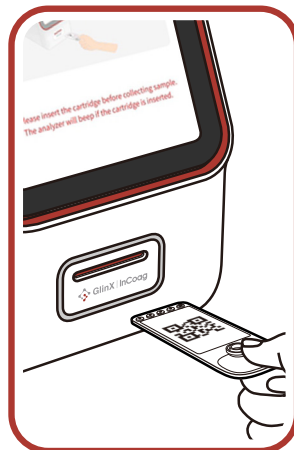


**Insert Test Cartridge**

- ① Turn on the device;
- ② Allow the test cartridge to reach room temperature for 10 minutes if removed from cold storage;
- ③ Before adding the sample, insert the cartridge horizontally into the lot with the QR code facing up;
- ④ The analyser will beep when the cartridge is inserted.



**Enter Sample Information**

- ① Sample ID and Species are required, other fields are optional;
- ② Tap the 'Next' button.

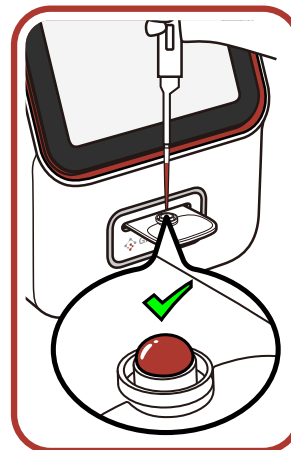

**Add Sample**

- ① Use a pipette to add 100  $\mu$ L of the sample;
- ② Allow the drops to gently fall into the sample well.



**Run Test**

- ① Tap the 'Start Testing' button;
- ② After the measurement is complete, view the results on the screen.

