


# Quick User Guide For Citrated Whole Blood Testing

**InCoag**  
Automated Coagulation Analyzer

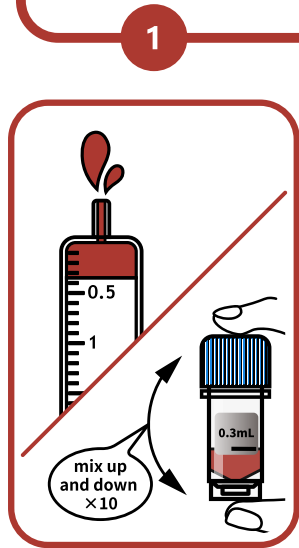

## Sample Preparation

- 1 Collect a minimum of 300µl of venous blood;
- 2 Within 30 seconds, transfer the sample to the citrate tube;
- 3 Gently invert the tube 10 times to mix;
- 4 Citrated blood is stable for 1 hour at room temperature and up to 4 hours at 2-8°C.




## Insert Test Cartridge

- 1 Turn on the device;
- 2 Allow the test cartridge to reach room temperature for 10 minutes if removed from cold storage;
- 3 Before adding the sample, insert the cartridge horizontally into the slot with the QR code facing up;
- 4 The analyzer will beep when the cartridge is inserted.



## Enter Sample Information

- 1 Sample ID and Species are required, other fields are optional;
- 2 Tap the 'Next' button.

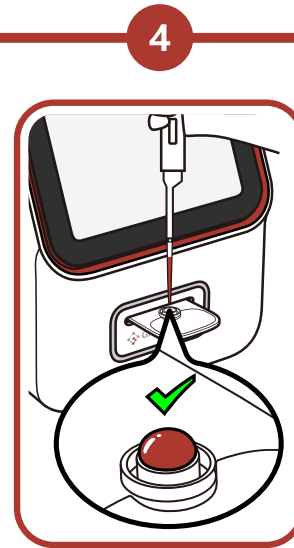

## Add Sample

- 1 Use a pipette to add 100µl of the sample;
- 2 Allow the drops to gently fall into the sample well.



## Run Test

- 1 Tap the 'Start Testing' button;
- 2 After the measurement is complete, view the results on the screen.

