Front

English

RELAXIN

English

RELAXIN

I. GENERAL INFORMATION

VECTAScan® Rapid Relaxin is intended to determine pregnancy and foal gestation in the bitch and queen, as well as to distinguish between pseudopregnancy and true gestation in the bitch and queen. The test measures relaxin, a hormone secreted by the placenta, which is present in significant amounts of this hormone is a reliable indicator of pregnancy. The VECTAScan Rapid Relaxin test provides an easy, reliable and rapid way to determine success or failure of planned matings or unwanted oocytes.

VECTAScan Rapid Relaxin is a simple test that uses the combination of two anti-Relaxin antibodies to quickly identify this hormone in biological samples (serum or plasma) from the bitch and queen. Colored gold particles sensitized with anti-Relaxin antibody bind to Relaxin molecules present in the sample. The formed complex migrates along a membrane. The complex is then captured on a semi-permeable membrane (sorbent and/or antibody) where its accumulation causes the formation of a clearly visible pink/red band. A built-in control line ensures that the test was performed correctly.

II. SAMPLE INFORMATION

- The test can be performed on serum or plasma (anticoagulated with EDTA, sodium citrate or heparin).
- Samples should always be collected with a sterile needle and syringe.
- Hemolysis does not significantly interfere with the test, but strongly hemolyzed samples must partly obscure a weak positive line.
- IV. SAMPLE STORAGE

Samples should be tested immediately after collection but not longer than 4 hours after collection. If stored at room temperature, freezing is further advised. Samples should be stored below -16°C (7°F) and kept frozen for up to 6 months. Samples should be thawed before use.

V. KIT CONTENTS

- 5 Samples capillaries
- Chase buffer dropper (bottle)
- Instructions for use only

VI. PRECAUTIONS

- Do not use components after expiration date.
- Store the test kit at 4°C - 25°C. Do not freeze. Storage at elevated temperatures (>47°C) can adversely affect test performance.
- Use the test immediately after opening the sealed pouch (within 10 minutes).
- Avoid touching or damaging membranes in windows of test device.
- The device should be placed on a flat, horizontal surface while performing the test.
- Use a separate pipette for each sample.
- Hold pipette and saline buffer bottle vertically when dispensing sample and chase buffer, respectively.
- Handle all reagents and samples as biohazardous material.
- For veterinary use only.

Note: Do not use test kit components after expiration date. The bands are only visible for quality control purposes. The dye does not interfere with the test results and will wash away while the test is developing.

TEST PROCEDURE AND RESULTS

Important: allow sample and chase buffer bottles to sit onto membrane in round sample port. Do not touch pipette tip or chase buffer bottle tip directly to the membrane.

1. SAMPLE APPLICATION

- See user's device panel and place the test device on a flat, horizontal surface.
- Remove the cap from the buffer bottle, transfer two drops of sample to the round sample port.

2. BUFFER DISPENSING

- Check that the sample has fully absorbed into the membrane.
- Remove the cap from the buffer bottle, hold it vertically and add two drops of buffer to the sample port.
- Leave the test device flat during formation of sample/reagent complex through the reading.

3. READING TEST

At 10 minutes, observe the presence or absence of pink/red test and control lines next to the T and C.

Note: It is possible to read the test within 10 minutes of visible pink/red test AND control lines have formed.

The presence of only a control line (next to the C) prior to the end of the development time (10 min) does not mean that the test is complete, as weak samples may appear slower than the control line.

4. RESULTS

Validation

The test is validated if a pink/red control line is present in each reading window. No control line means the test is invalid and the results should be repeated.

Interpretation

- Formation of test line and control line sample is positive.
- Control line only (no test line): sample is negative.

Positive Samples

All positive results confirm pregnancy.

Negative Samples

A negative result means that Relaxin is not present. A negative result may be due to a false negative or to a false positive. To be detected (only for test in pregnancy status), two negative results at least one week apart may be required for confirmation of pregnancy. A negative result, especially when there is a history of mating or mating, is not conclusive. Non-pregnancy can be confirmed completely only starting from day 27 after birth's ovulation (31 days post-mating in queen).

Note: A result should always be interpreted in the context of all available clinical information and history for the animal being tested.

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